

NEW

HEMOHUB[™]
INTELLIGENT DATA
MANAGER

Centralized management for Hemostasis testing

HEMOSTASIS INNOVATION IS HERE. ►

 **Instrumentation
Laboratory**
A Werfen Company

Unmatched operational performance and laboratory decision-making, all from a single workstation

Today's Hemostasis laboratories are challenged to do more with less. HemoHub Intelligent Data Manager enhances the efficiency of the Hemostasis testing operation, improving workflow and decision-making—and ultimately, patient care.



Connects all IL Hemostasis and ROTEM® testing systems and allows users to centrally:


- View critical results and analyzer status in real time
- Manage analyzer quality control simply and efficiently
- Support laboratory decision-making
- Optimize operations and facilitate accreditation

Centralized information management for flexibility, workflow efficiency and better patient care.

Make the connection with HemoHub.

View critical results and analyzer status in real time


HemoHub offers real-time notification of critical results to help ensure the best patient care.

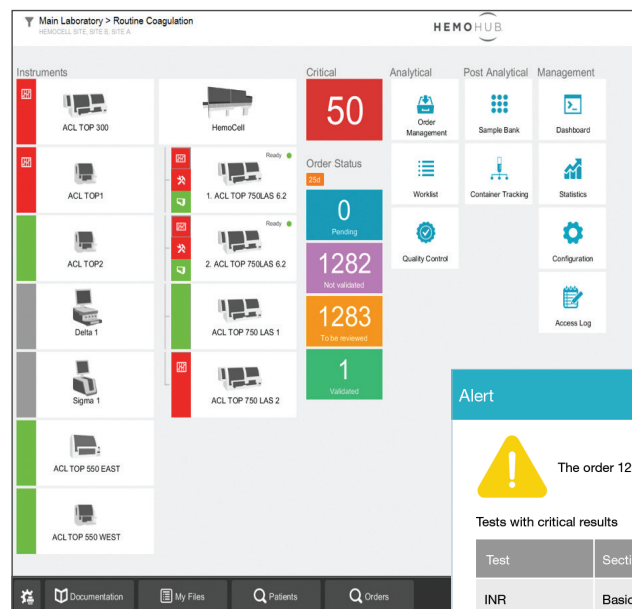

Home screen dashboard with critical result alerts.

Monitor critical results

- Flags samples with critical results requiring urgent review with red indicator
- Records all actions taken on critical results for full traceability

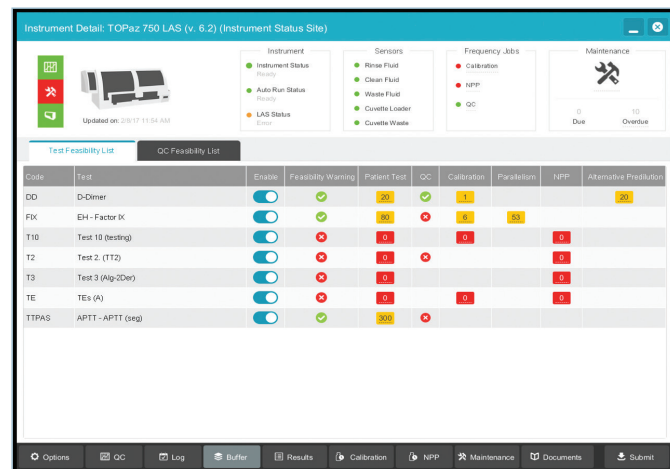

View analyzers' QC status across departments.

View status of all analyzers, anytime

- Real-time connectivity status
- QC monitoring ensures quality is maintained

Increase productivity and facilitate management of instruments*

- Instrument, sensor, frequency job and maintenance status monitoring
- Centralized viewing of test and QC feasibility with user-defined warning limits

Minimize workflow interruptions with flagged sample alerts

- Indicators for redraw or retest, which help minimize delays in patient management decisions
- Alerts for analyzer or sample issues

Experience true scalability


No limit on number of analyzers connected

Optimize workflow for greater efficiency, simply.


*ACL TOP Family 50 Series systems only.

Manage analyzer quality control simply and efficiently

HemoHub allows QC management from one central location to ensure quality results from all analyzers. Ultimately, this means time savings and improved quality management.

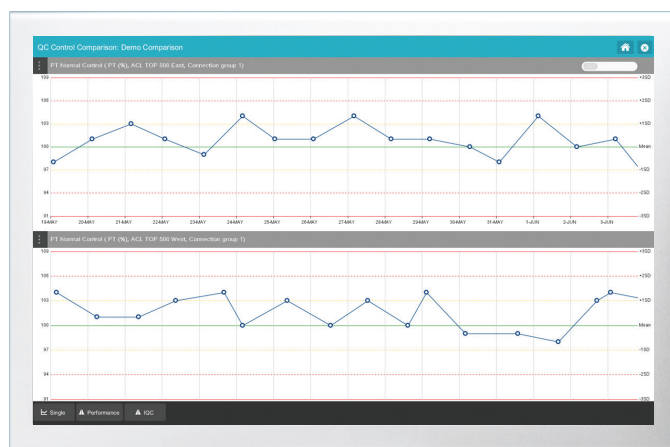

View QC data from different analyzers, side by side on a single graph.

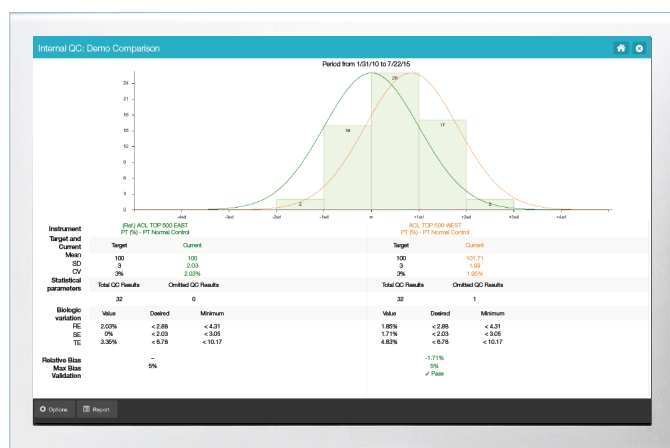

Compare internal QC parameters.

The screenshot displays the 'QC report in Excel' interface. It shows a detailed table of QC results for multiple analyzers. The table includes columns for 'Analyzer', 'Lot', 'Material', 'Target', 'Current', 'Control Limits', and 'Status'. The data is organized into multiple rows, each representing a different analyzer and its corresponding QC results. The table is presented in a standard Excel format with a grid and various formatting options.

View QC report in Excel.

Compare analyzer QC

- View QC data from multiple IL analyzers
- Compare QC performance among multiple IL Hemostasis systems in single or multiple Levey-Jennings charts
- Set up current and alternate QC lots, indicate “Active” lots and “Inactive” controls

Compare internal QC

- Set one analyzer as a reference to compare and evaluate key metrics
 - Mean value
 - Relative bias
 - Maximum bias

Manage QC for any IL Hemostasis analyzer from a single screen

- Auto-upload QC material lot information and expiration date

Generate QC reports

- Create PDF and Excel files, and export data for AccuTrak


Monitor QC continuously with real-time alerts

- Set QC rules
 - Specify conditions and actions for QC
 - Trigger actions, including QC omission
- Set QC alerts
 - Set user notification for out-of-range QC and expired QC material

Enhance operational performance for quality results, easily.

Support laboratory decision-making*

Each patient is unique. HemoHub allows evaluation of results within the context of patient history, to help deliver the best patient care.

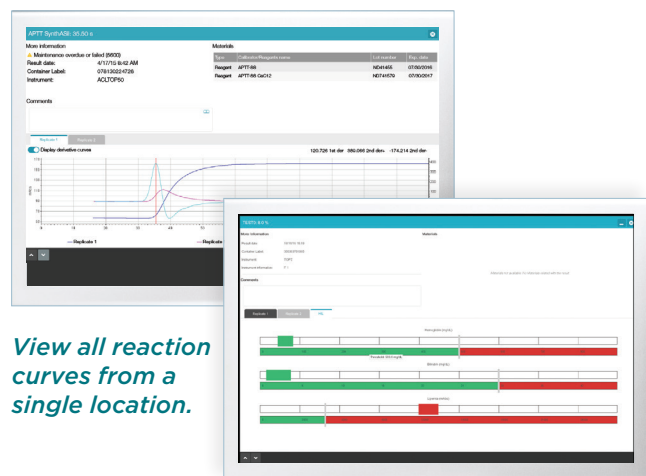

Centralized view of clot signature curves

- Access reaction curves for all connected IL analyzers from a single interface

Pre-analytical sample checks**

- Detect assay-specific interfering substances
 - Flag samples exceeding thresholds for Hemolysis, Icterus, Lipemia
- Alert users to under-filled sample tubes with tube-fill height detection
- Prompt operator to check for clots with abnormal sample aspiration

SMITH, TOM	15/07/14 1234567	15/07/14 1234568	14/07/14 1234569
Coagulation			
APR	5.50	5.55	5.50
APR Coagulation Study			
TP - TP Activity	9		12
PT - PT (ratio)	70.00		130.00
PT - PT (sec)	11.0		13.0
APTT - APTT (ratio)	0.85		1.00
APTT - APTT (sec)	26.0		36.0
Fibrinogen-derived	150.0		111.0
EH - Factor VW functional	68.0	125.0	115.0
Coagulation - Hemorrhage Study			
ET - Factor VIII	Rejected	Rejected	Rejected
EH - Factor VIII Cr	Rejected	Rejected	Rejected
EH - Factor IX	Rejected	Rejected	Rejected
EH - Factor XI	Rejected	Rejected	Rejected
EH - Factor XII	Rejected	Rejected	Rejected
Total			
Export			

View patient-results history and trending reports for a complete picture of Hemostasis.

12345678 - 456789	SMITH, INGRID	Urgent	Interpretation	Value
Type	Inpatient			
Collect Center	Laboratory			
Doctor	Not stated			No data
Service	Laboratory			
Diagnosis	Without diagnosis			
Test	Result	Deviation	Interpretation	Value
APR	5.50	APR	5.80 - 1.20	demo
APR Coagulation Study				
PT Activity	11	%	9 - 12	-15 %
PT (ratio)	70.00	ratio	70.00 - 130.00	-40 %
PT (sec)	12.0	seconds	8 - 14	-15 %
APTT (ratio)	1.10	Ratio	0.80 - 1.20	10 %
APTT (sec)	26.0	seconds	25.1 - 36.5	13 %
Fibrinogen-derived	150.0	mg/dL	100.0 - 600.0	15 %

Automatic rerun and reflex testing save staff time.


Flexible rule configuration based on multiple parameters

- Configure rules around patient history and parameters (e.g., age, gender, location)
- Set rules to automatically rerun or reflex tests
- View results of multiple reruns side by side for faster decision-making


Customized patient information supports laboratory decisions, quickly.

Optimize operations and facilitate accreditation


HemoHub includes centralized management of analyzers and test results to ensure quality and efficiency.

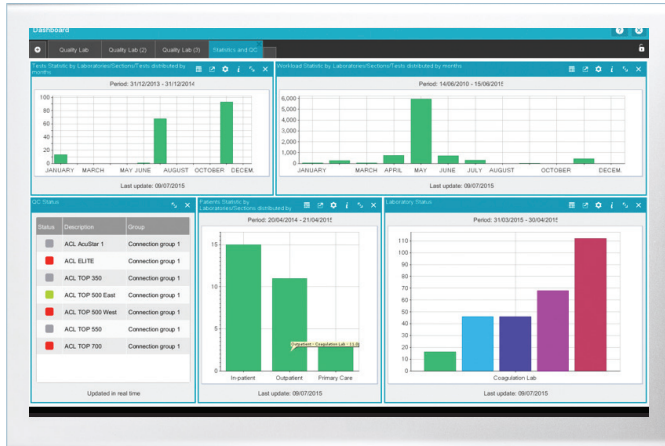

Customize KPI dashboards for specific performance indicators.

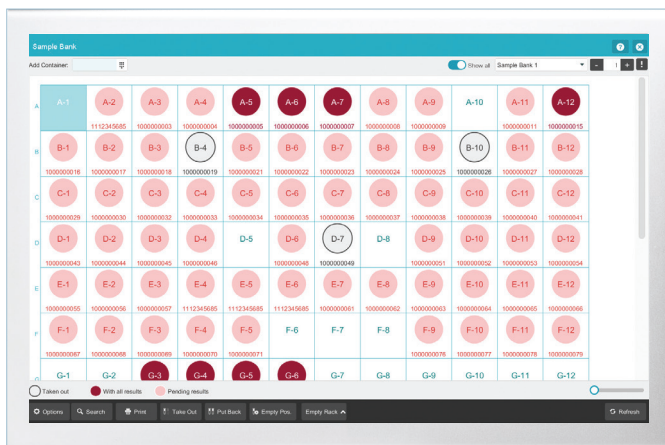

Track specimen containers.

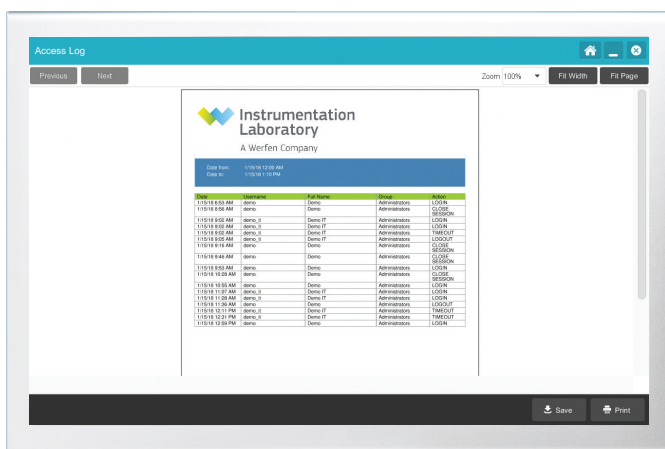

View user-access log.

Data-driven decisions

- Run daily statistics, patient and critical values reports
- Run QC documentation
- Generate work-list reports
- Configure and monitor 25+ key performance indicators (KPIs), including turnaround time

Simplify sample-bank inventory

- Efficiently manage aliquots and primary tubes paperlessly

Manage test orders, efficiently

- Access patient and test orders easily
- Create work lists remotely

Integrate accreditation support

- Audit log documents all user actions
- User-access rights provide information regarding user competency and compliance
- Document management system maintains all documentation in one convenient, secure location

Centralized management for superior quality management.

What makes it unique?

Flexibility to meet specific lab operational needs

- One centralized interface for management of multiple networked sites
- Scalable for an unlimited number of patient files and analyzer connections
- KPI-configurable dashboard
- 100+ configurable statistical report formats

Efficiency to optimize workflow

- Simple rerun and reflex processing
- Sample alert flags
- Simplified sample-bank inventory

Quality results for better patient care

- QC management from one central location
- Easily accessible patient-result histories
- Patient-parameter configured rules
- Integrated accreditation support

HemoHub is an essential component of HemoCell Specialized Lab Automation


HemoCell is a workcell solution, exclusively for Hemostasis testing, to increase throughput, reduce turnaround time and ensure pre-analytical quality

- Higher throughput
- Shorter turnaround times
- Reduction of pre-analytical issues
- Centralized control of Hemostasis testing
- Retention of Hemostasis expertise in your lab
- Lower operator exposure to biohazards
- Low implementation impact in the laboratory



Enhanced performance, workflow and laboratory decision support

HemoHub Intelligent Data Manager standardizes and optimizes Hemostasis testing across systems and networks—for flexibility, efficiency and better patient care.

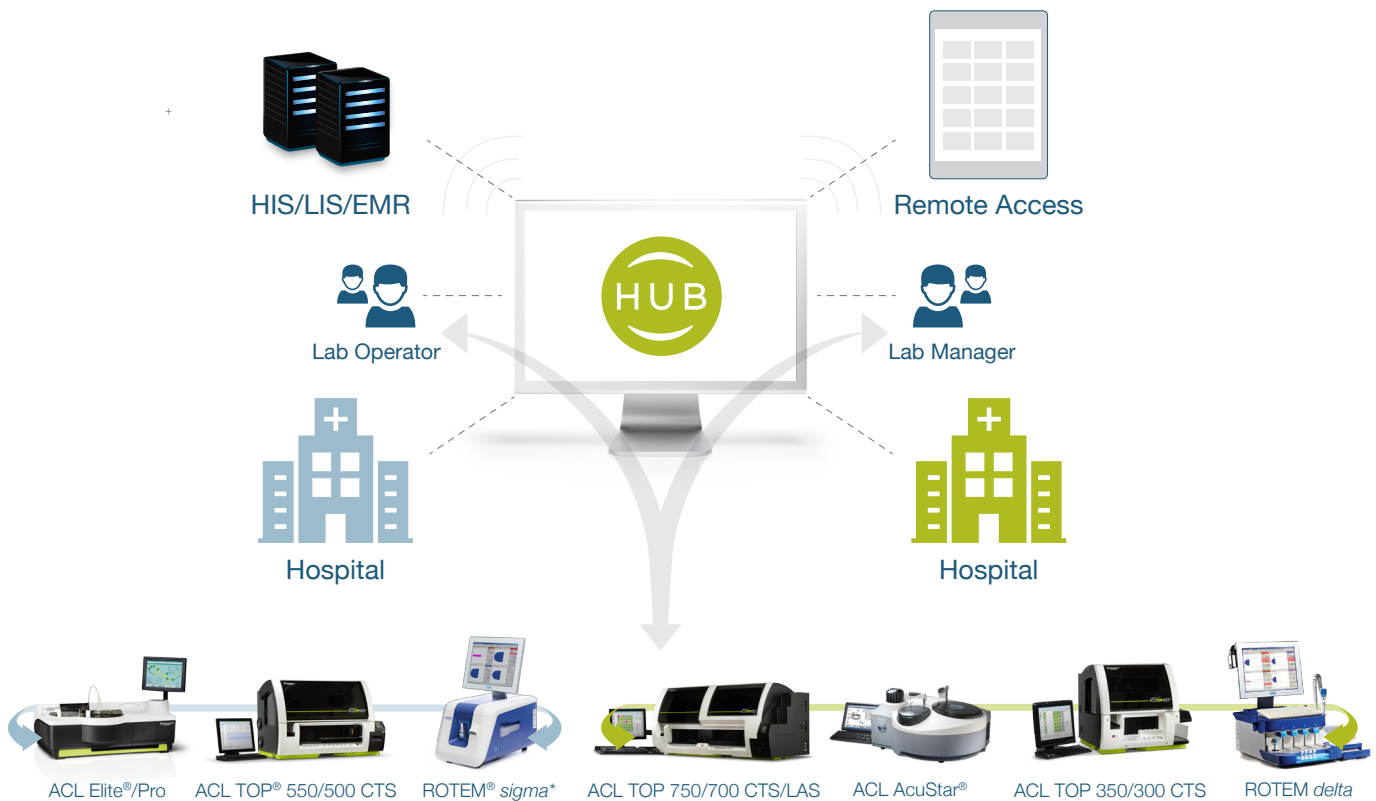

*Not FDA 510(k)-cleared or Health Canada-licensed.

Werfen Corporate Headquarters

Plaza de Europa, 21–23
08908 L'Hospitalet de Llobregat
Barcelona, Spain
+34-93-4010101
werfen.com

Instrumentation Laboratory Headquarters

180 Hartwell Road
Bedford, MA 01730 USA
+1-781-861-0710
instrumentationlaboratory.com

Worldwide Locations

The Americas

Brazil
São Paulo
+55-11-46227878
br.werfen.com

Canada
Richmond Hill, ON
+1-800-552-2025 x6115
instrumentationlaboratory.com

Colombia
Bogotá
+57 15 221 052

Mexico
Mexico City
+52-55-5262-1760
mx.werfen.com

Uruguay
Montevideo
+5982-481-81-33

USA
Bedford, MA
+1-781-861-0710
instrumentationlaboratory.com

Asia-Pacific

Australia
Sydney
+61-02-9098-0200
au.werfen.com

China
Beijing
+86-10-59756055
Hong Kong
+852-2792-7773
Shanghai
+86-21-66308671
cn.werfen.com

India
New Delhi
+91-490-29-550
in.werfen.com

Japan
Tokyo
+81-3-5419-1301
jp.werfen.com

Korea
Seoul
+82-1899-9217
kr.werfen.com

Thailand
Bangkok
+66-271-226-28/9

Europe

Austria
Vienna
+43-1-256-58-000
at.werfen.com

Belgium
Brussels
+32-2-7252052
benelux.werfen.com

Czech Republic
Prague
+420-246-090-931
cz.werfen.com

France
Paris
+33-182-30-86-00
fr.werfen.com

Germany
Munich
+49-89-909070
de.werfen.com

Hungary
Budapest
+36-1-882-73-10
hu.werfen.com

Italy
Milan
+39-02-25221
it.werfen.com

Lithuania
Kaunas
+370-37-313157
lt.werfen.com

The Netherlands
Breda
+31-76-5480100
benelux.werfen.com

Poland
Warsaw
+48-22-336-18-00

Portugal
Lisbon
+351-214247312
pt.werfen.com

Russia
Moscow
+7-499-124-45-59
ru.ilwerfen.com

Spain
Barcelona
+34-902-20-30-90
es.werfen.com

UK
Warrington, England
+44-1925-810141
uk.werfen.com

For all other countries visit
international.werfen.com

The Instrumentation Laboratory logo, ACL, ACL AcuStar, ACL Elite, ACL TOP, HemoCell, HemoHub, HemosIL, ReadiPlasTin, RecombiPlasTin, SynthAFax and SynthASil are trademarks of Instrumentation Laboratory Company and/or one of its subsidiaries or parent companies and may be registered in the United States Patent and Trademark Office and in other jurisdictions. All other product names, company names, marks, logos and symbols are trademarks of their respective owners.

©2018 Instrumentation Laboratory. All rights reserved.